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Read all the instructions carefully before use

Definition of use

Medical Device for fastening of UROMED urine leg bags.

User

Physicians (medical professionals), nursing staff and appropriately trained patients/ relatives.

Patient Group

Adults and children using a urine bag for urine drainage.

Indication

The UROMED single leg pant is an unsterile medical device for fastening of UROMED urine leg bags on the leg of patients using (bladder/kidney/male external) catheters to drain urine.

Contraindication

Do not use the device if you have circulatory and/or sensory disorders in the legs or known thrombosis of the leg veins. Do not continue to use the device if allergy or hypersensitivity occurs.

In the case of suprapubic urinary drainage, please ask your doctor whether one-leg pants are suitable.

Clinical Use

To maintain mobility when using a urine leg bag and improvement of quality of life.

Application

Use the sizing table to choose a suitable size.

Pull the holder up over the preferred foot into the desired position on the leg. Close the belt around your stomach or waist.

Open the slip pocket for the holder and slide the leg bag into the pocket.

Insert the T-Tap Outlet through the opening located in the middle of the lower end. Make sure that the T-Tap Outlet is closed.

Ensure that qualified personnel teach you how to use the device correctly.

Pre-treatment and preparation

Check the device for damage after each wash.

Do not use a damaged product.

Do not use the device after the expiry date.

⚠ Warnings

Do not wear the device on injured skin.

Choose a suitable size to prevent vascular compression and secondary health problems.

Please consult your doctor if you are not sure whether the product is suitable for you.

Report any serious incident that has occurred in relation to the device to UROMED Kurt Drews KG and the competent authority in your country.

Storage and handling conditions

Store the device at room temperature, dry, dust-free and protected from light.

Disposal

After use, the medical device may present a biological hazard.

Dispose of the device after use in accordance with local regulations and safety rules (e.g. with household or hospital waste).

Technical description and performance features

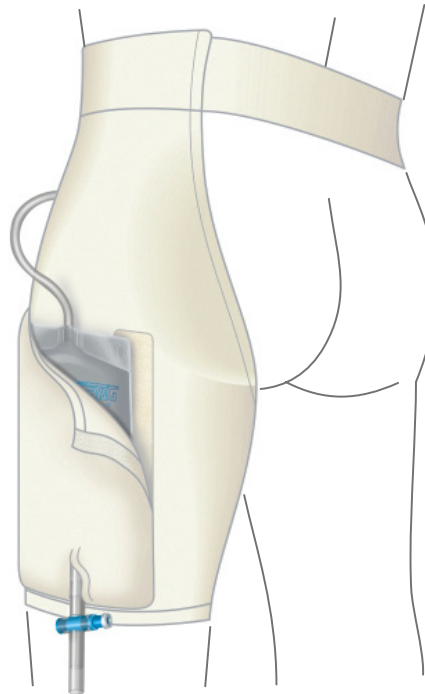
The UROMED single leg pant can be combined with all UROMED Urine Bags up to a maximum filling quantity of 750 ml.

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