***iQ*CATH**

IQ2604.xx

IQ2614.xx

Gebrauchsanweisung	Deutsch	2
Instructions for use	English	4
Notice d'utilisation	Français	6
Istruzioni per l'uso	Italiano	8
Gebruiksaanwijzing	Nederlands	10
Bruksanvisning	Svenska	12
Brugsanvisning	Dansk	14
Käyttöohje	Suomi	16
Bruksanvisning	Norsk	18
Notkunarleiðbeiningar	Ísland	20
Kasutusjuhised	Eesti	22
Instrucciones de uso	Español	24
Instruções de utilização	Português	26
Οδηγίες χρήσης	Ελληνική	28
Használati utasítás	Magyar	30
Instrukcja użytkowania	Polski	32
Návod k použití	Čeština	34
Instrucţiuni de utilizare	Română	36
Инструкция за употреба	Български	38
Navodila za uporabo	Slovenščina	40
Upute za uporabu	Hrvatski	42
Uputstvo za upotrebu	Српски/srpski	44
Uputstvo za upotrebu	Bosanski	46
Упатство за употреба	Македонски	48
دستور العمل استفاده	فارسی	50
قسطرة التبول الخارجي ذاتية الالتصاق	العربية	52

Deutsch

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English

Hydrophilic Single-use catheter for ISC (Intermittent self-catheterisation)

- Ready for use
- Catheter with 2 functional zones (SAUER-rounded tip and guiding zone)
- Urine collection bag
- Total length: ca. 430 mm, Length without funnel: ca. 400 mm
- 4 sizes: CH 12, CH 14, CH 16, CH 18

Group of Users: IQ2604.xx, IQ2614.xx– Men

Intended use:

The IQ-Cath is a single use product designed for temporary drainage of urine from the urinary bladder in a person with bladder voiding dysfunction for therapeutic or diagnostic purposes. It is suitable for intermittent self-catheterisation (ISC) or carer-assisted clean intermittent catheterisation (CIC).

Indication:

The indication for ISC or CIC is made following relevant diagnostic tests and is dependent on the individual's situation.

The IQ-Cath is suitable for catheterisation in the following situations: bladder voiding dysfunction of various origins (e.g. neurogenic), underactive detrusor or acontractile detrusor, chronic urinary retention, abnormal micturition (straining) with or without residual urine.

Contraindication:

The IQ-Cath may be unsuitable in the following cases, and a health care professional must be consulted before use: injury of the urethra, bleeding of the urethra, acute inflammation of the urethra or prostate, obstructions (e.g. tumours) in the urethra, at the sphincter or at the prostate blocking passage.

Instructions for use:

The first use should be supervised by a professional.
If liquid has already leaked from the saline sachet into the packaging, the catheter may no longer be used.

Risks and Cautions:

The use of catheters (especially improper use) may cause irritation/infection of the urinary tract. You must seek medical attention if this occurs.
Improper use may cause residual urine to remain in the bladder. This catheter is intended for single use only. The risk of infection increases if the catheter is reused or reprocessed. Do not use without activating with sterile saline solution 0.9% first as it is not lubricated. Tap water affects catheter sterility and thus poses an increased infection risk. Do not use if the sterile packaging or the catheter is damaged as there is a risk of infection. We recommend an operating temperature range of 15-25°C.
Any serious adverse event occurring during use of IQ-Cath must be reported to the manufacturer and the competent authority of the user's country of residence.

Catalog number

Serial number

Store in a dry place

This way up

Do not resterilise

Quantity

Keep away from sunlight

Sterilised with ethylene oxide

Single sterile barrier system

Single sterile barrier system with protective packaging outside

Batch number

Observe instructions for use

CE marked medical device

Use by (year-month-day)

Not for reuse

Pharmaceutical central no.

Drug Tariff

List of products and services

Do not use if package is damaged

Date of manufacture

Manufacturer

Swiss authorised representative


SAUER-rounded tip

Ready for use

Ready for use


Fig. 1-7: Avoid touching any part of the catheter surface (except for the connector) directly.

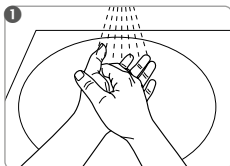
Before use, wash hands thoroughly and disinfect them.


Open the water sachet by folding and squeezing.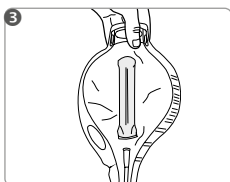


Make sure the whole content of the sachet is emptied into the packaging and all of the catheter surface is activated by the fluid for at least 30 sec.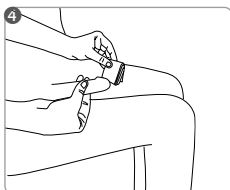


Clean the opening of the urethra and the surrounding area with a disinfectant suitable for mucous membranes (observing the instructions of the manufacturer of the disinfectant).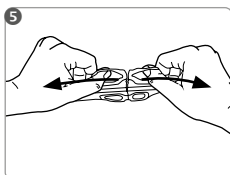

Open the urine bag at the front end and release the tip of the catheter.
If the catheters sticks to the packaging please pull out twisting it slowly at the same time.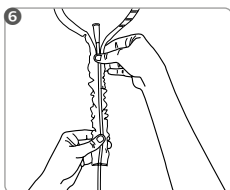

Gently insert the catheter into the urethra until the urine begins to flow. When the urine stops flowing, pull the catheter out slowly. You can use the functional packaging as a gripping aid.

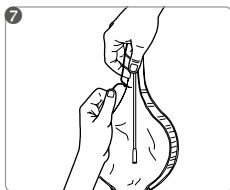Push back the catheter into the urine bag and seal this by making a knot. To empty the bag tear at the back end.

Dispose of the catheter according to local authority guidelines.

MANFRED SAUER GmbH does not accept responsibility for damages or injuries that may result from improper handling contrary to these instructions for use or from neglecting the hygiene guidelines.

Should you have any questions or problems, please turn to a medical professional or to our customer adviser: MANFRED SAUER · Neurott 7-15 · 74931 Lobbach · Germany · +49 6226 960100

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